



Clinical Disinfectant Wipes 200 Wipes



USE:

Intended for disinfecting patient shared equipment and medical environments

DESCRIPTION

Cleaning and disinfecting wet wipes for equipment and surfaces. For use where there is a high risk of infection.


FEATURES & BENEFITS

- Passes EN 13727, EN 16615, EN 13624, EN 16615, EN 14476, EN 16777 & EN 14348
- Alcohol and PHMB free
- Kills bacteria and effective against viruses such as Coronavirus
- Dermatologically tested
- Efficient solution release, excellent solution retention low linting and strong

HOW TO USE

Use PPE as required. Use one wipe to disinfect, holding the wipe flat and wiping the surface in an S-shape ensuring the entire surface is wet. Use a fresh wipe if your wipe becomes soiled or dry. For visibly soiled surfaces, use one wipe to remove visible soil prior to disinfection. Do not use on sensitive surfaces to quaternary ammonium compounds. Allow the surface to dry naturally in line with stated contact times. Once opened use within 3 months. This product is not intended for end point sterilisation of medical devices.

Microorganism	Contact Time	Microorganism	Contact Time
Bactericidal Efficacy		Virucidal Efficacy	
EN 13727	1 min	EN 14476	1-2 min
EN 16615	1 min	EN 16777	1 min
Yeasticidal Efficacy			
EN 13624	1 min	Mycobactericidal Efficacy	
EN 16615	1 min	EN 14348	1 min

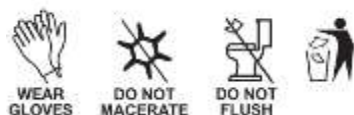

TYPICAL PRODUCT DATA

Appearance: Wet nonwoven wipe material
 Colour: White
 Odour: Slight chemical odour
 pH: 11.0-12.0

PACKAGING

BA071
 1 x 200 wipes

